

COVID-19

RAPID TEST KIT FOR IgM/IgG



- PEACE OF MIND
- IMMEDIATE RESULTS
- RESULTS WITHIN 10-15 MINUTES




- Significant time and cost saving over lab methods
- Identifies infection before symptoms
- Assists in the control of viral transmission
- Easy to use and read, with no specialist equipment required

RAPID POINT-OF-CARE DIAGNOSIS TEST OF CORONAVIRUS INFECTION.

The new rapid IgM-IgG combined antibody test* for COVID-19 is now available to test for the detection of IgG and IgM antibodies of coronavirus from a small specimen of human serum, plasma or whole blood. The detection works with proteins to indicate if a person's immune system has responded with the COVID-19 virus to indicate infection.



- Disinfect finger with alcohol pad
- Gently, but firmly prick finger with Lancet
- Collect blood with a dropper
- Add 1 drop of fingertip blood into the sample well
- Add 2 drops of sample dilution into the sample well slowly
- Wait between 10-15 minutes, Read results based on chart provided

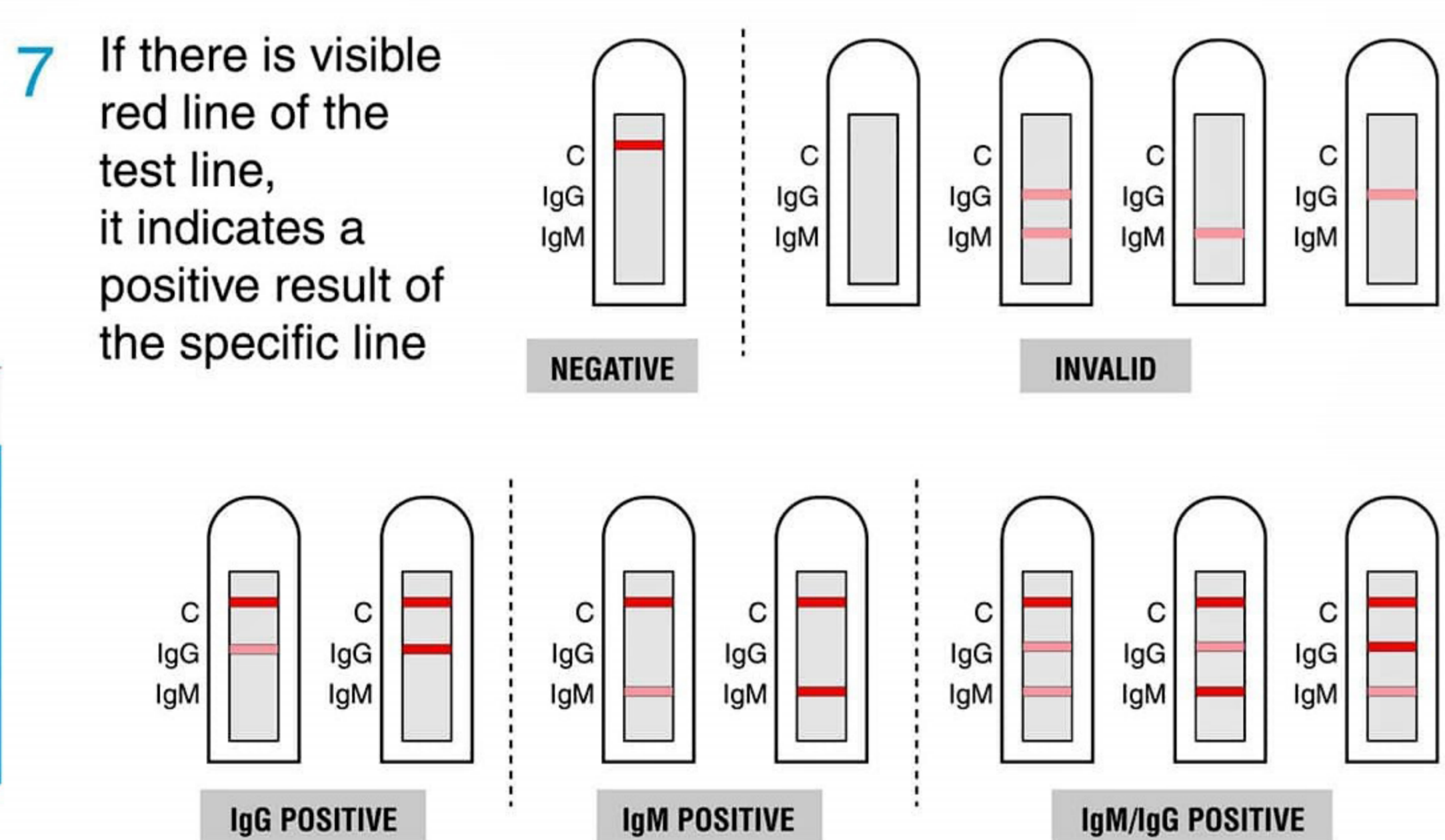

The test can be used for rapid screening of patients with or without symptoms of the virus. A small droplet sample of blood taken from the patient and placed with a dropper onto the test kit can show results in around 15 minutes. The test can help to determine if a person has been infected with COVID-19 even after the virus is no longer present.

The test is ideally suited for hospitals, clinics and can also be effectively deployed in businesses, schools, airports, seaports and train stations, etc.